

ICEROBOTICS

Attaching the IceTag to the rear leg of the cow

Important: The strap should not be attached to the leg of the cow too tightly. Once attached, it should be possible to insert a few fingers between the cow's leg and the IceTag. A video demonstration can be found at <http://www.youtube.com/watch?v=gP2Hwr0YdOE>

1




Ensure that the IceTag is positioned with the arrow on the strap pointing upwards. Wrap the strap around the cow's leg. The strap should be loose enough to insert a few fingers between the cow's leg and the IceTag.

2



Line up the appropriate hole with the outer attachment pin.

3

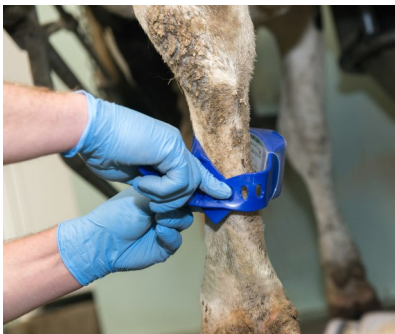

Twist the strap and insert the longer (lower) end of the pin first

4



Move the strap round to allow the shorter end of the pin through the hole

5



Line up the strap with the second pin and feed any excess strap through the guide


Removing the IceTag from the cow's leg

1



Pull the end of the strap through the guide and free from the inner pin. Angle the end of the strap downwards

2

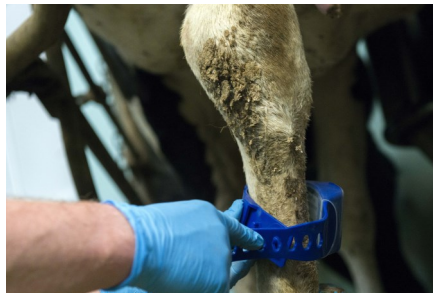

Twist the strap round to remove the shorter end of the pin from the hole, then the longer end.